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Contents

1. Reprocessing process

(Basic, Manual, AER & Ultrasonic, EndoRapid, Disinfection)

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
4. OER-AW 사양

5. OER-AW 기능

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1. Reprocessing process : Basic concept

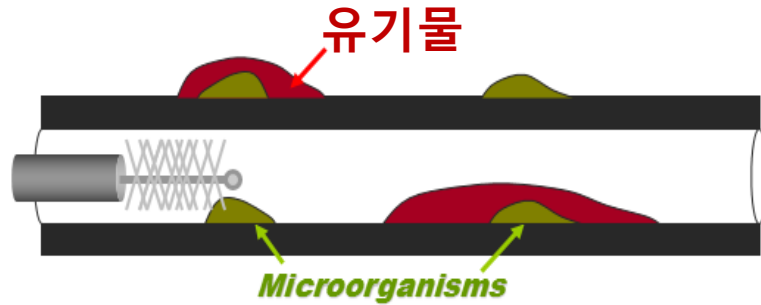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

Flushing all the channels at once

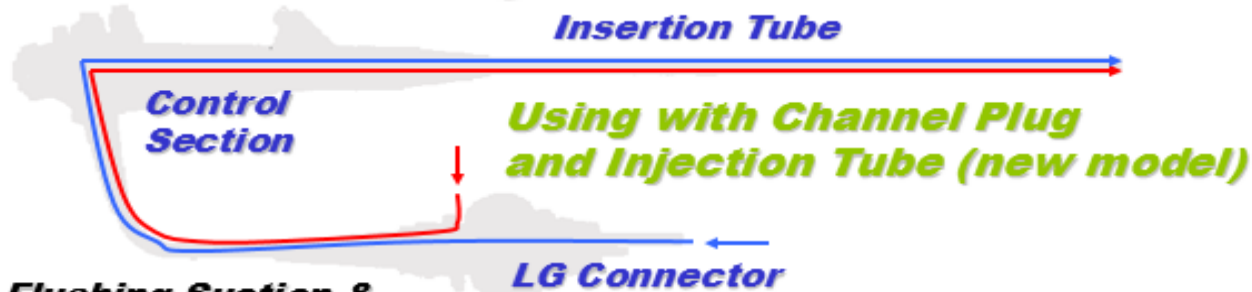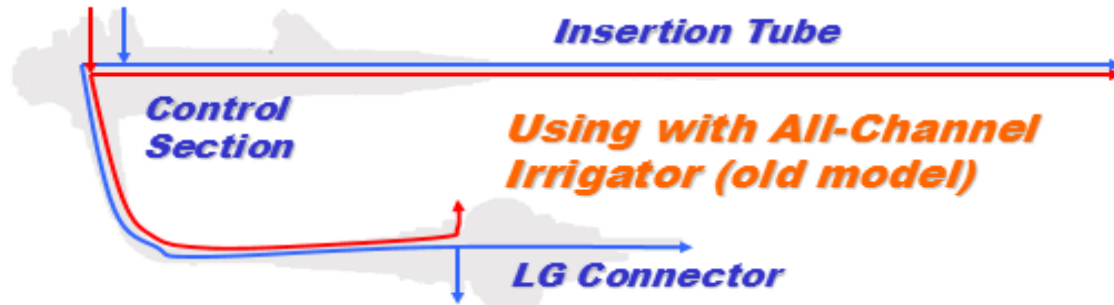Flushing Suction & A/W channels separately

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

1. Reprocessing process : AER

Automatic Endoscopic Reprocessing (AER)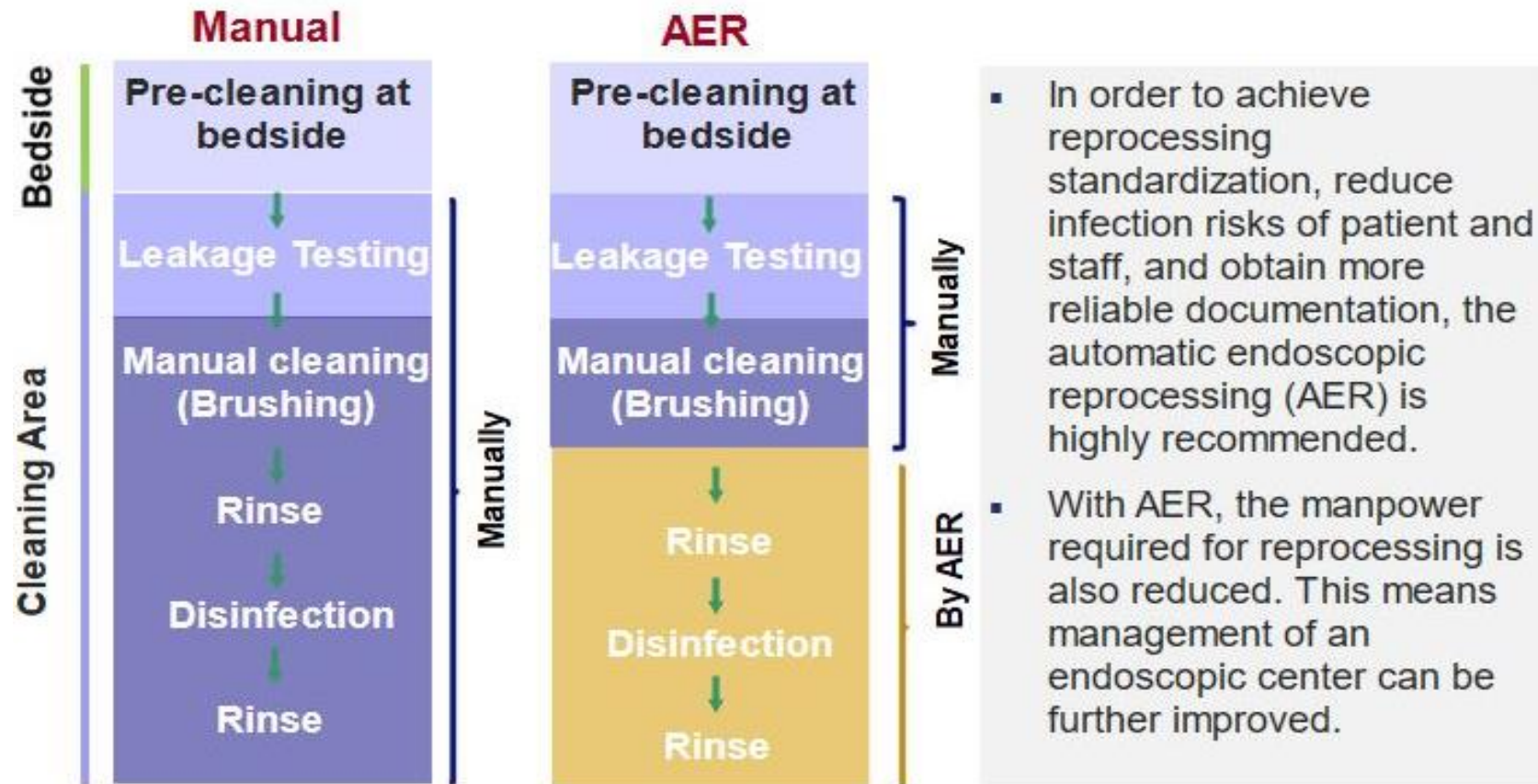

1. Reprocessing process : OER

Modified Manual Cleaning Using OER-AW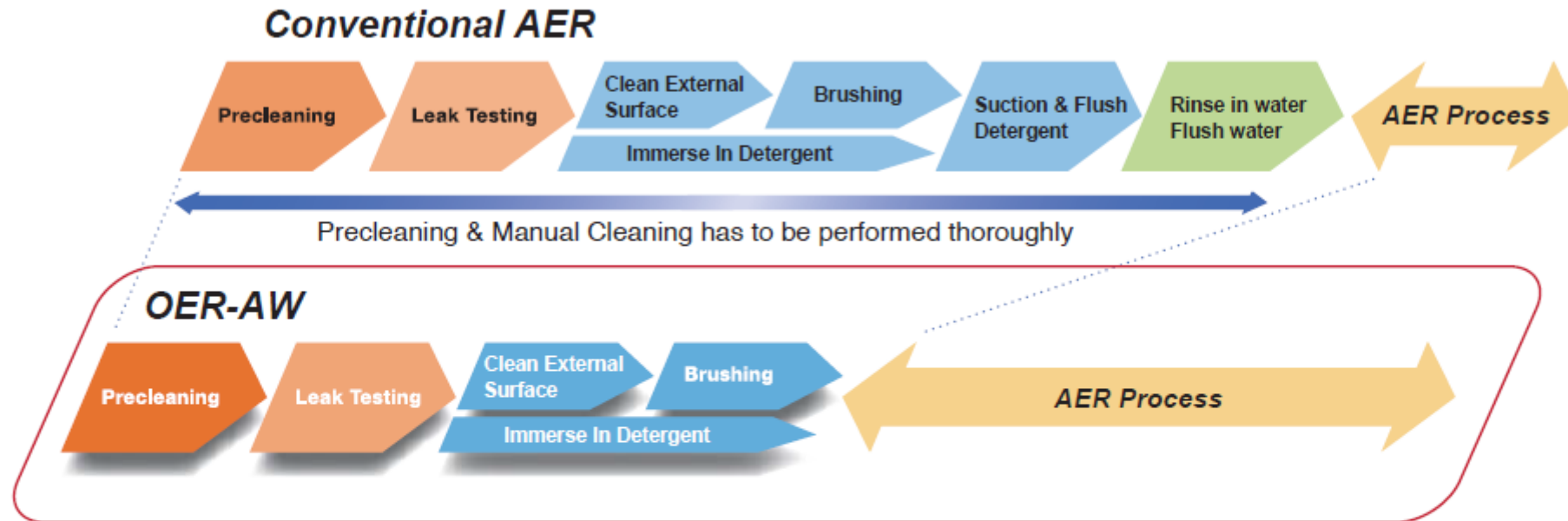


With OER-AW, manual cleaning steps after brushing can be skipped.*

*As for the endoscopes with elevator, additional steps are required after brushing. Please see the OER-AW Operation Manual.


OER-Pro, the equivalent product of OER-AW for the US market is cleared by FDA that “use of the OER-Pro enables the user to perform modified manual cleaning of the endoscope prior to automated cleaning and high-level disinfection in the OER-Pro.”

1. Reprocessing process : **AER(OER-AW)**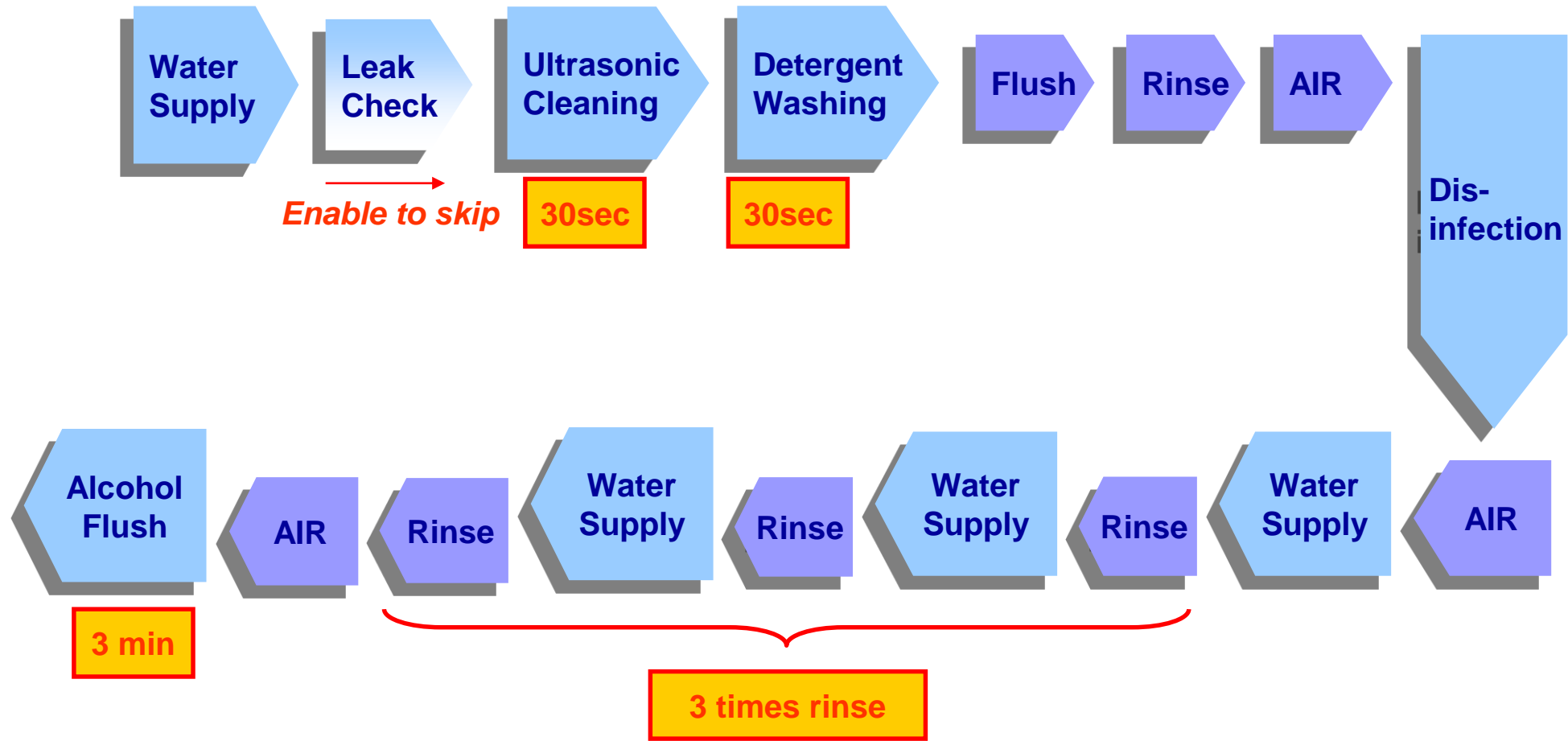

1. Reprocessing process : **Ultrasonic Cleaning**

Ultrasonic & Detergent Cleaning of OER-AW & OER-A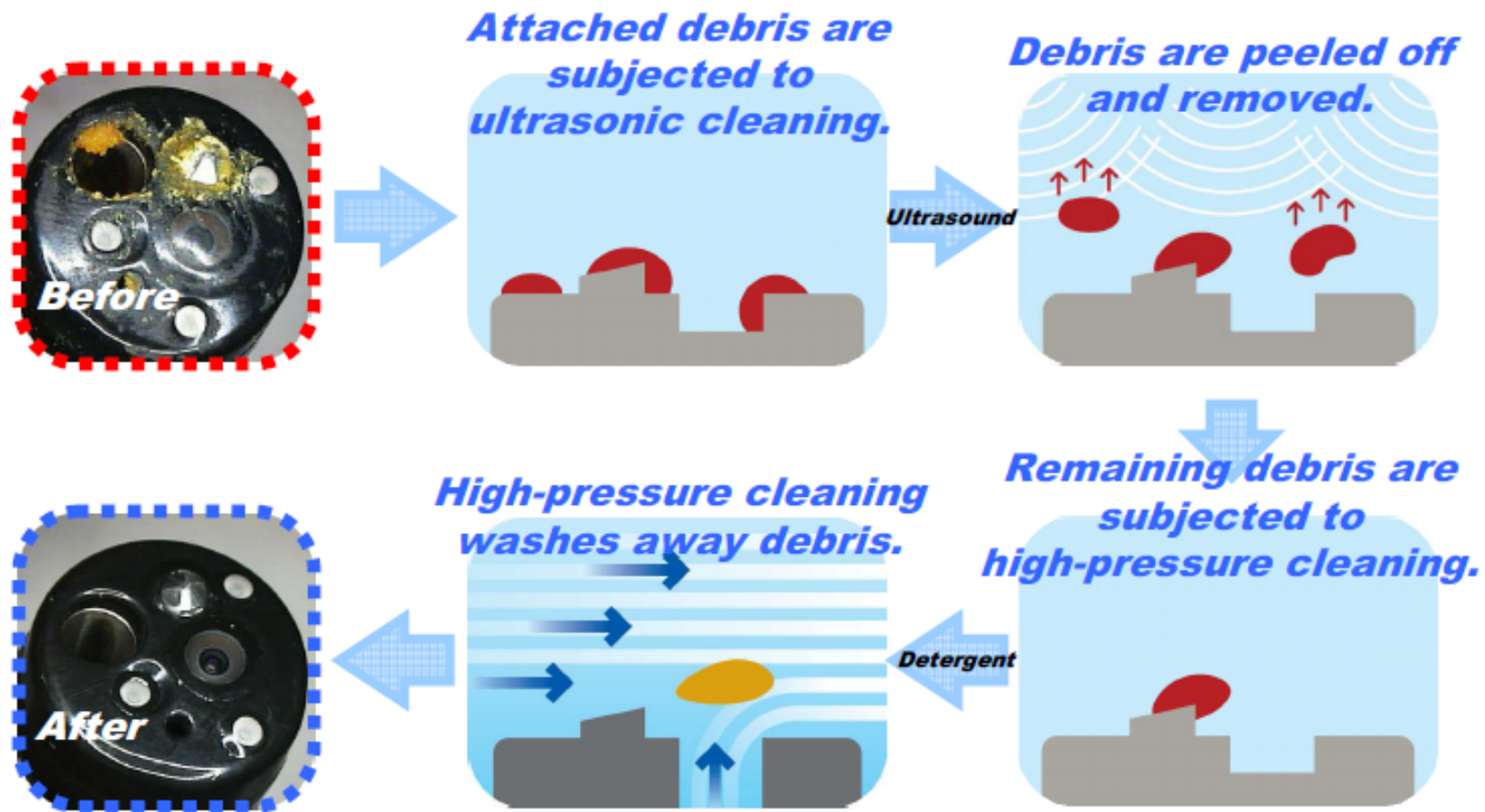


1. Reprocessing process : **Ultrasonic Cleaning**

Ultrasonic & Detergent Cleaning

Combination of ultrasonic & detergent makes it possible for users to perform “Modified Manual Cleaning”


1. Ultrasonic Cleaning (30sec)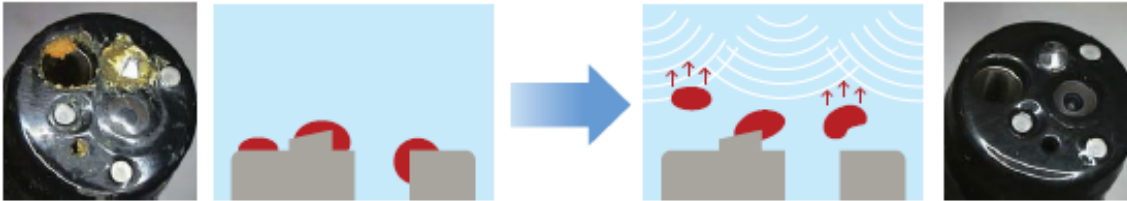

Debris on the endoscope is loosened and removed by ultrasonic cleaning

Ultrasonic transducers are placed under the cleaning tub.

Insertion tubes are set up on the transducers so that the outer surface of the insertion tubes and universal cords can be reprocessed sufficiently.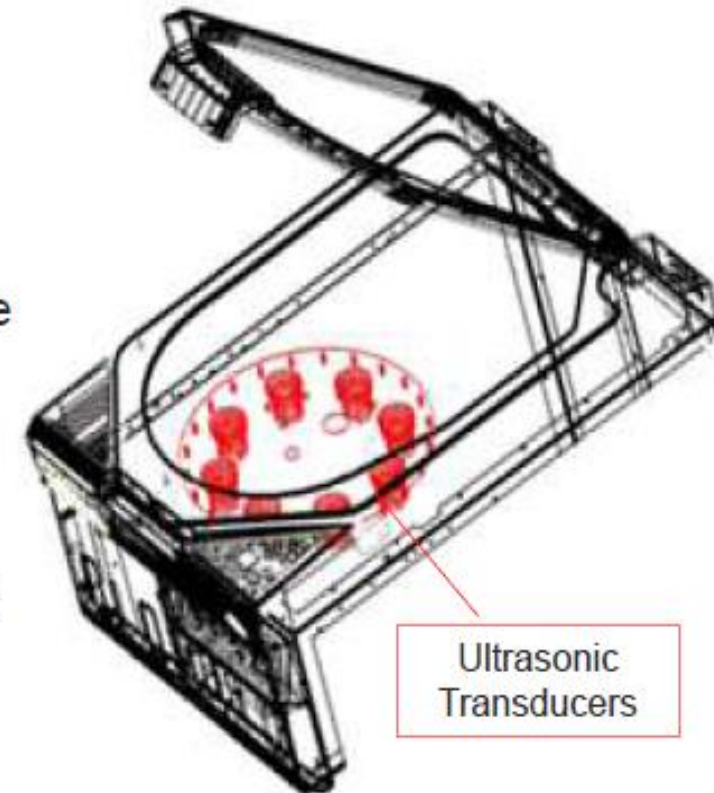

Ultrasonic Transducers

1. Reprocessing process : **Ultrasonic Cleaning + EndoRapid**

Alkaline detergent, EndoRapid (초음파세척기용 전용세척제)


- Dedicated detergent for OER-AW
- Alkaline detergent (Potassium Hydroxide : 수산화칼륨)
- Superior cleaning power in a short time at room temperature
- Low forming

Ultrasonic & Detergent Cleaning


Combination of ultrasonic & detergent makes it possible for users to perform "Modified Manual Cleaning"

1. Ultrasonic Cleaning (30sec)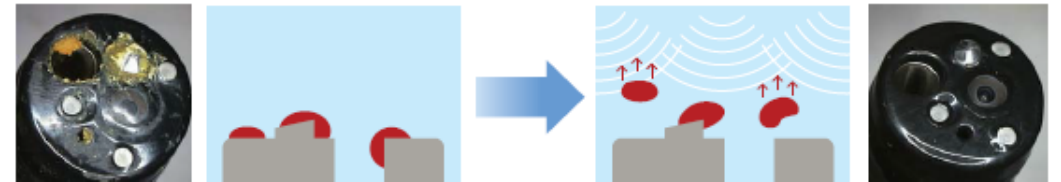

Debris on the endoscope is loosened and removed by ultrasonic cleaning

2. Detergent Cleaning (30sec or more)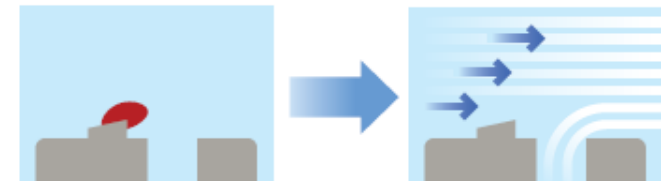


High-pressure cleaning washes away the remaining debris using water-jet & detergent

1. Reprocessing process : **Ultrasonic Cleaning + Endorapido**

Difference between Enzyme detergent and Alkaline detergent


【Method】

- Product to be cleaned: PTFE tube (20cm)
- Degree of contamination: As indicated by ISO/TS 15883-5 Annex L
- Fluid flushing cleaning for 30 sec. by OER-3 (the same type as OER-AW)
- Evaluation method: After staining with residual protein detection liquid, recover the staining solution combined with protein to use alkaline solution, and determinate its absorbance

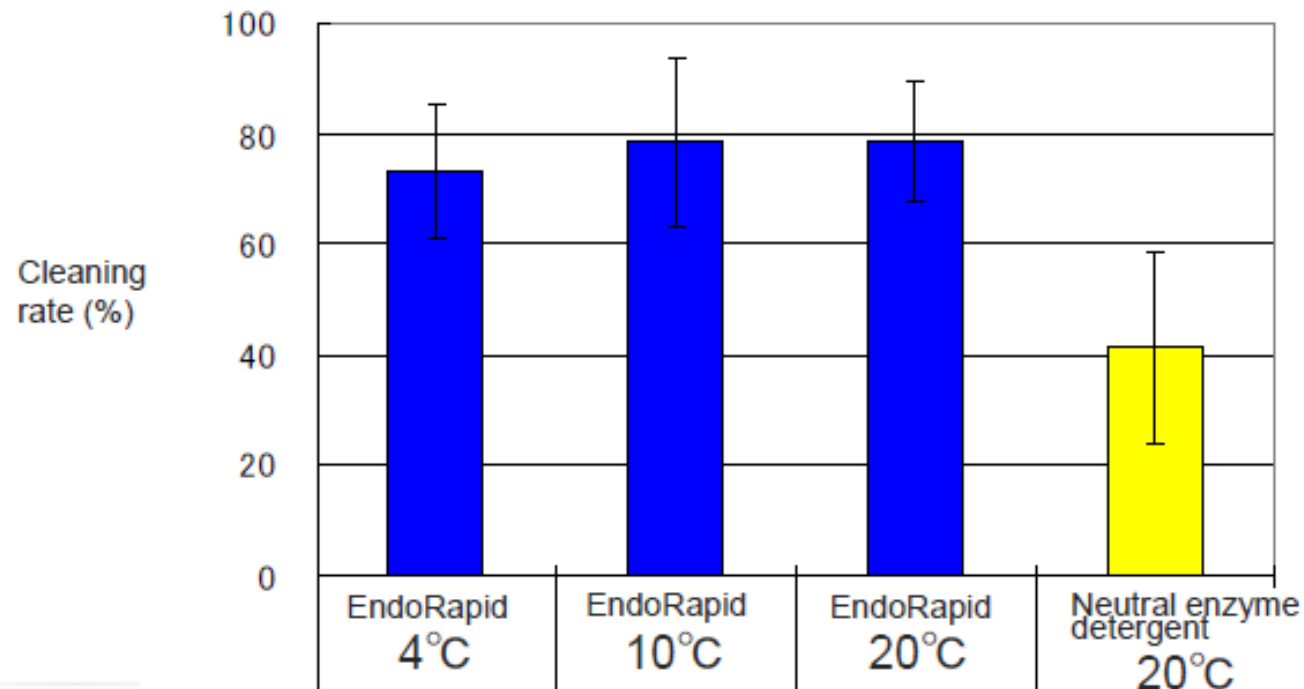1. Reprocessing process : **Disinfection**

Decomposition product of Peracetic acid

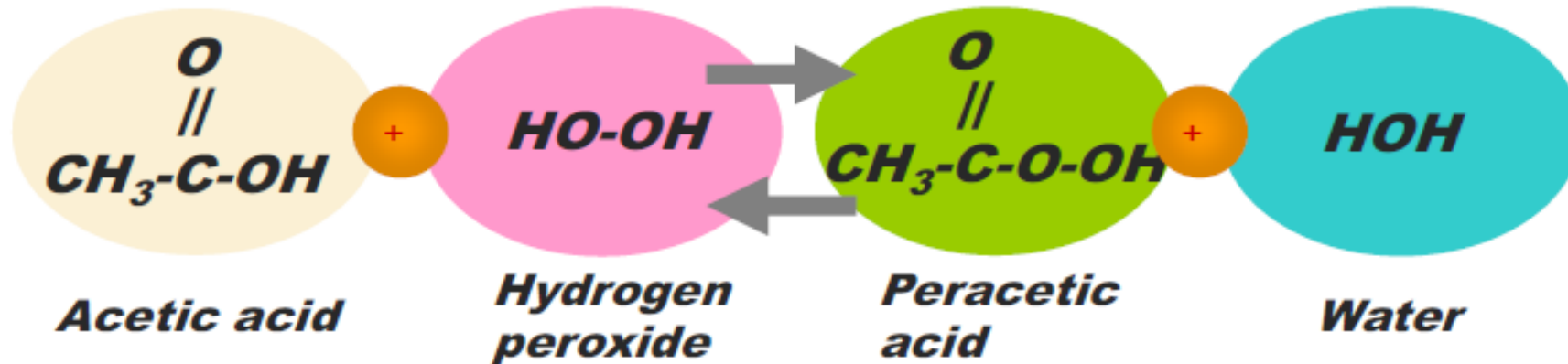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OER-AW Self disinfection function

The OER-AW reprocessor itself can be disinfected automatically.

- Internal fluid lines are automatically disinfected during every cycle.
- Water line disinfection process is also available for optional process (this process should be performed on occasions such as when the internal 0.2 micron water filter is replaced, or before using the OER-AW after more than 14 days without use.).

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